


TAMPER EVIDENT

Rainbow Trays™



Colour coded Anaesthetic trays

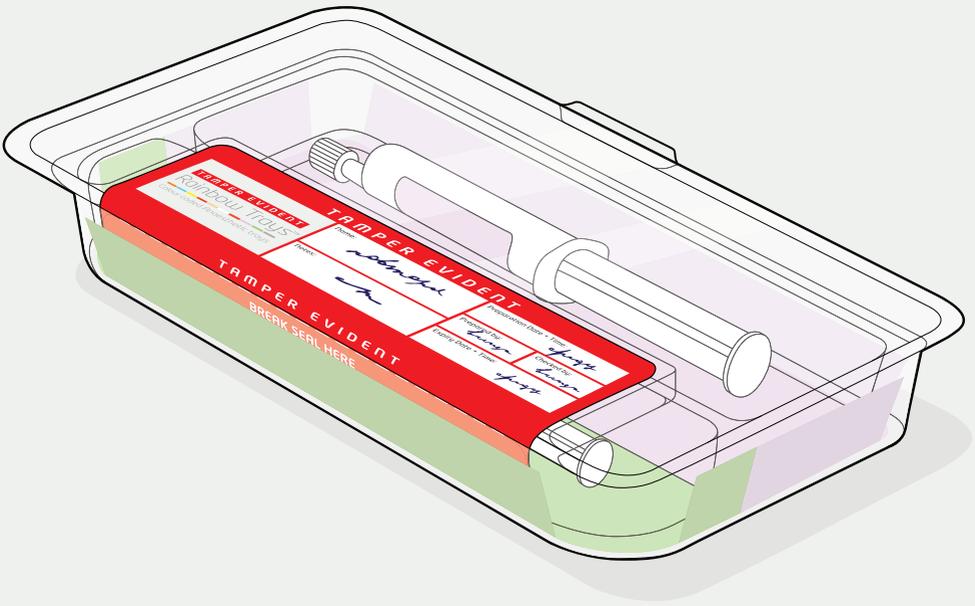


INSTRUCTIONS FOR USE

LOADING THE TRAY


1

Place the tamper evident tray into the base tray

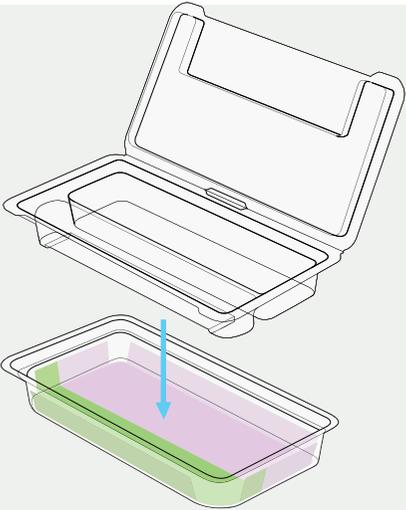


2

Place syringes into colour coded compartments

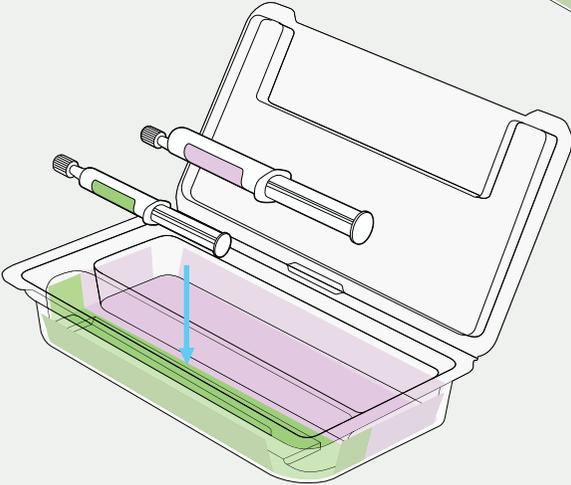

3


Click tray shut

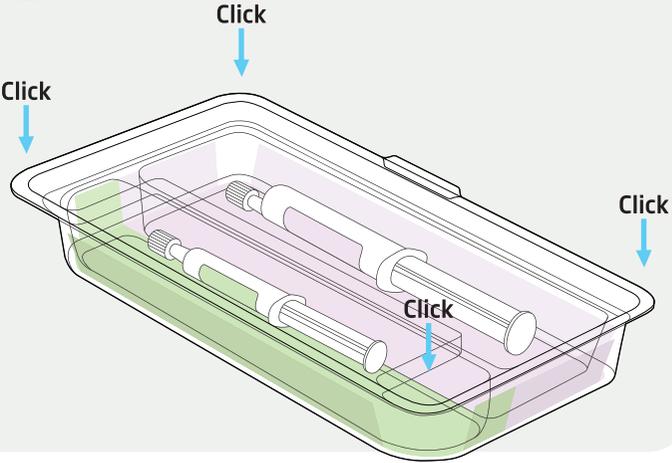

SECURING THE TRAY

4

Fill in the label

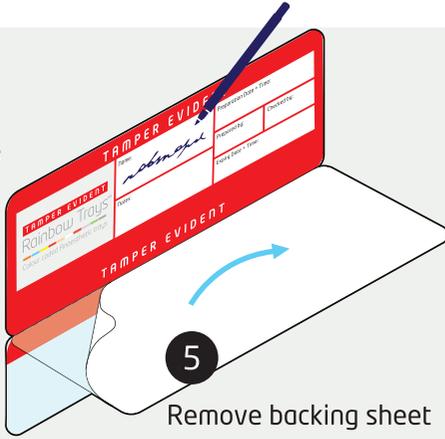


5

Remove backing sheet


6

Stick the adhesive flap over the text

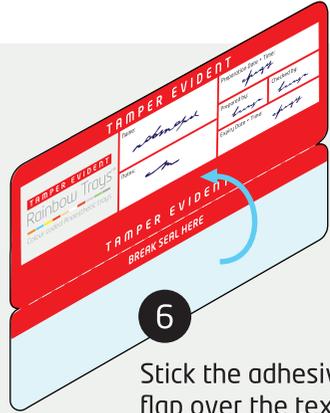


7

Remove the tray from the base and align the label with the top edge of the square area of the tray

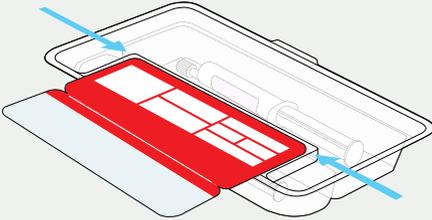


8

Wrap label around the tray and stick to underside

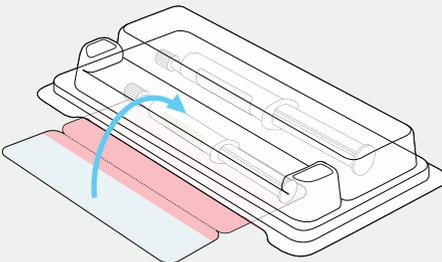

9



Place the tray back into the base. Your anaesthetic drugs are now secure in the tamper evident tray and ready for use

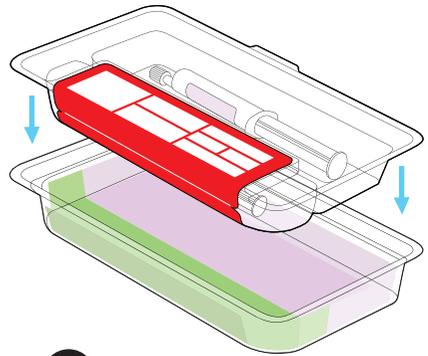

USING THE TRAY

10

Remove the tamper evident tray from the base using the tab

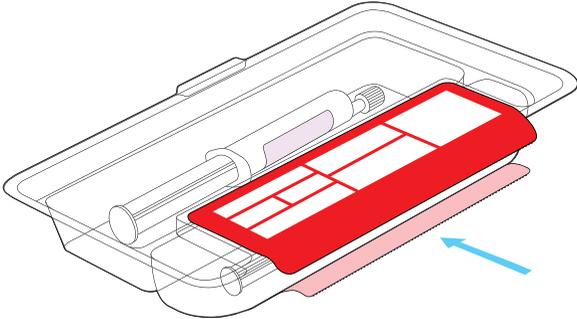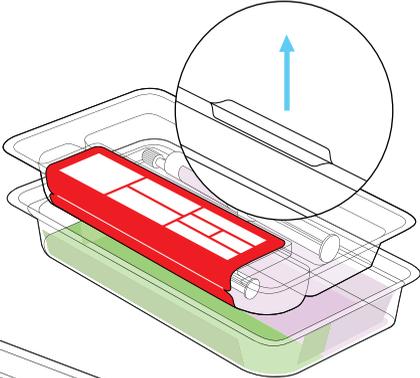

11

Break the seal in the middle of the perforations

12

Open lid – ready for use

